

Recommended Debonding Protocol

Use conventional debonding pliers to remove the brackets. To remove any remaining adhesive from the concave lingual surface of the incisors, use an ovoid carbide finishing burr.



Use pliers to debond



Debond anteriors



Remove excess adhesive from incisors using finishing burr



Remove excess adhesive from molars using finishing burr



CONTENT CONTRIBUTED BY:

Woo-Ttum Bittner is a specialist in orthodontics who graduated from the Humboldt University of Berlin, Germany. He is the founder of ADENTICS®, a group of orthodontic practices that focus on lingual treatment. Mr. Bittner is President of the 2015 German Lingual Society (DGLO). He is a chief contributor to the suressmile Lingual Clinical Protocol. As the first suressmile user for lingual in Europe, he has treated over 500 suressmile lingual cases.